Securafix®

Seite 2 - de: Gebrauchsanweisung
Page 3 - en: Instructions for use
Page 4 - fr: Manuel d'instruction
Página 5 - es: Instrucciones de uso
Pagina 6 - it: Istruzioni per l'uso
Strana 7 - bs: Uputstvo za upotrebu
Pagină 8 - ro: Instrucțiuni de utilizare
Sayfa 9 - tr: Kullanma kılavuzu
Strona 10 - pl: Instrukcja użytkowania
Стр. 11 - ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended user

Dentist

Composition

Qualitative data:

Powder: Calcium aluminium fluorosilicate, polyacrylic acid, iron oxide

Liquid: Tartaric acid, water

Quantitative data:

Powder: 75 % calcium-aluminium-fluorosilicate, 25 % polyacrylic acid

Liquid: > 6 % polyacrylic acid, > 90 % water

Intended purpose of the product

Securafix is a glass ionomer cement and thus has an adhesion to enamel and dentin.

Indications

Fixation of crowns, bridges, inlays, onlays, posts and orthodontic bands.

Contraindications

Do not apply directly to dentin close to the pulp. Do not use in patients with known allergy to the ingredients.

Performance characteristics of the product

Securafix hardens and chemically bonds to enamel and dentin.

Specific precautions

None.

Application

Clean enamel, dentine and metal surfaces thoroughly and dry them perfectly. Shake up the powder before removing it! Apply Securafix at room temperature (15-23°C). At higher temperatures, the use of a cooled glass plate is advantageous. Powder and liquid are added to a glass plate or mixing block and the powder is mixed in portions. For 1 level measuring spoon of powder, use 2 drops of liquid. The mixing time is about 30 seconds. Securafix can be processed at room temperature for about 1'20" minutes and then sets in the mouth in about 3'00" minutes. Higher temperatures shorten, lower temperatures extend the processing time. For narrow bridges and crowns, make sure that there is not too much excess of Securafix and that the viscosity is not too high, as glass ionomer cements have a specific flow behaviour. Securafix has the right consistency when it can be applied to the tooth stump and into the crown with a hard brush. Avoid water and saliva ingress during the setting time.

Picture warnings**Residual risks**

The working instructions and precautionary measures described in the package leaflet accompanying the product must be observed in any case. If an adverse reaction occurs in the patient or other adverse events occur (despite following these instructions), we request that you inform us in every case, describing the accompanying circumstances as precisely as possible.

Supplementary notes:

Rinse used metal instruments immediately with cold water before the cements harden. Cement residues after removing orthodontic bands can be removed quickly and gently with an ultrasonic scaler. Close the powder container tightly immediately after use. Keep medicines out of the reach of children.

Side effects

If there is insufficient residual dentine thickness (e.g. heavily ground dies) without insulation or if there is a large excess of cement, irritation of the pulp may occur. Undesirable side effects of this medical device, when the device is properly processed and used, are extremely rare. If you become aware of any adverse effects – even in doubtful cases – please inform us and describe the accompanying circumstances and symptoms as precisely as possible.

Risks in case of overdose are not known.

Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage

Store in a dry place. The Product should be stored between 0° C - 22°C.

Minimum shelf life

Do not use after the expiry date.

Variants

REFWP4110 - Securafix, A2, 24,0 g powder / 10,0 g liquid

REFWP4093 - Securafix, A3, 7,0 g powder / 5,0 g liquid

REFWP4146 - Securafix, 10,0 g liquid

Disposal

Dispose of unused material according to regulations (hazardous waste).

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

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Willmann & Pein GmbH