Light Bloc®

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Page 3 - en: Instructions for use

Page 4 - fr: Manuel d'instruction

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Strana 7 - bs: Uputstvo za upotrebu

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Instructions for use

Patient target group

None, as Light-Bloc does not come into contact with the human body.

Intended users

Dental technicians

Intended purpose of the product

Light-Bloc is a light-curing acrylic resin for model preparation in dental technology. The modellable plastic material is supplied in syringes.

Indications

The Light-Bloc material is used in dental technology for model preparation. It is used for compensating intersecting sections and to build up and supplement prepared stumps and models.

Application

Press the paste out of the syringe, remove with standard laboratory instruments and apply to the model or stump. Light-Bloc Paste is modelled and smoothed with a brush or instrument. When applying Light-Bloc Paste, the plaster must no longer be damp. A UV light source should be preferred for polymerisation, though an LED can also be used. After curing, the material can be processed with a milling cutter.

Side effects

No undesirable side effects are known.
There are no known risks of overdose.

Interactions

There is no known interaction of the product with other medical products, pharmaceuticals or other substances.

Storage

Light-Bloc Paste has a shelf life of three years if the material has not been stored above 22°C in its original container during this time. This also applies to packs that have already been opened and resealed.
Avoid direct sunlight.

Minimum shelf life

Do not use after the expiry date.

Variants

 WP5118	Light-Bloc GF, 1 x 2.0 ml
 WP5126	Light-Bloc GF, 2 x 2.0 ml
 WP5138	Light-Bloc Advanced, 1 x 3.0 g
 WP5139	Light-Bloc Advanced, 2 x 3.0 g

Disposal

Unused material can be cured and disposed of with household waste.
Uncured material should be disposed of like methacrylate waste.

Any serious incidents relating to the product must be reported to the manufacturer and the competent authority of the Member State where the user is based.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 41 23/9225-0
Fax: +(49) 41 23/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 41 23/9228-0
Fax: +(49) 41 23/9228-49
www.wp-dental.de
info@wp-dental.de

Made in Germany

H224 Extremely flammable liquid and vapour.

H317 May cause allergic skin reactions.

H315 Causes skin irritation.

H373 May cause organ damage through prolonged or multiple exposure.

H360 May damage fertility or the unborn child.

H412 Harmful to aquatic life with long lasting effects.

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Variantes

REF WP5118	Light-Bloc GF, 1 x 2,0 ml
REF WP5126	Light-Bloc GF, 2 x 2,0 ml
REF WP5138	Light-Bloc Advanced, 1 x 3,0 g
REF WP5139	Light-Bloc Advanced, 2 x 3,0 g

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