D

Doublident®

Gebrauchsinformation

Additionsvernetzende Silikone (Vinylpolysiloxane) zur Modellabformung.

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Doublident®

Instructions for use

Addition silicones (vinylpolysiloxanes) for model duplication.

Doublident is an addition silicone for model duplication for the dental laboratory. The 1:1 mixing ratio between Base and Catalyst makes measuring and mixing easy. Doublident is suitable for mixing by hand or in a duplicating unit machine, because it has elevated fluidity and does not require a vacuum. Doublident reveals details precisely, it has long term dimensional stability and is resistant to contact with various types of resins, investments and gypsums.

Preparing the model:

Before using Doublident, check that the model to be duplicated is perfectly clean. If it needs milling, be sure to remove any oily residues; use of a solvent such as acetone is recommended. Wait for steam to evaporate, if used, and make sure that the model is perfectly dry. Select a flask of appropriate size, then anchor the model onto its base with wax.

Batching and mixing with manual and vacuum technique

1) Measure or weigh equal amounts of Catalyst (white) and Base (coloured) in a graduated container.
2) Mix the two components well, until colour is even.
Attention: Check that there are no un-mixed residues or strips of colour on the base and walls of the container.

Use

Slowly pour Doublident, preferably from a height of about 30 cm above the flask. It is not necessary to create a vacuum. If the quantity measured out is not sufficient to complete the duplication, wait until the silicone has hardened completely and then add more material in the same way. The material will bind perfectly and the addition will have no effect on the final result. Use a jet of compressed air between the model and the mould created. Do not use mechanical force with a knife or other implement, and do not pull it away from the flask, or you may deform the mould.

We recommend the use of Doublident at room temperature; low temperatures reduce the fluidity and enlarge the setting time, high temperatures reduce notably the setting time. For times of mixing, working and setting, see the values in the table below. Doublident is compatible with all types of gypsums, investments or poured resins.

Important recommendations

Stick to an exact 1:1 ratio to ensure that

timing is correct and the product's final properties are not altered. If using materials of different kinds such as die-spacers, resins, etc., you must wait for these products to stabilise completely to prevent a risk of inhibition of the polymerisation reaction. The surfaces with which the material comes into contact must be perfectly clean, grease-free and dry. Close bottles after use, and do not switch their lids.

Storage

Guaranteed for 24 months if stored correctly between 5°-27°C (41°-80°F).

Technical data

Automatic mixing time	30 sec.
Manual Mixing time	1 min
Working time	3 minutes
Setting time	<15 minutes
Dimensional change after 24 hours	- 0.05%

Processing and mixing time relate to room temperature 23°C

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