Fisseal®

Seite 2	- de:	Gebrauchsanweisung
Page 3	- en:	Instructions for use
Page 4	- fr:	Manuel d'instruction
Página 5	- es:	Instrucciones de uso
Pagina 6	- it:	Istruzioni per l'uso
Strana 7	- bs:	Uputstvo za upotrebu
Pagină 8	- ro:	Instrucțiuni de utilizare
Sayfa 9	- tr:	Kullanma kılavuzu
Стр. 10	- ru:	Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentist

Composition

Qualitative data:

Oligomeric methacrylates, sodium fluoride, camphorquinone, inorganic fillers

Quantitative data in % w/w:

Methacrylates 79, inorganic fillers 20, sodium fluoride 1

Intended purpose of the product

Fisseal is a light-curable composite for sealing fissures.

Indications

Fisseal is a light-curing composite with fluoride for fissure sealing. It is a sealant for occlusal surfaces for the purpose of caries prophylaxis.

Contraindications

The material contains methacrylates.

Do not use in patients with known allergy to the ingredients.

Performance characteristics of the product

The composite material cures by irradiation with light.

Specific precautions

None

Application

1. Clean the surface thoroughly and cover it with an etching gel (orthophosphoric acid), the blue colour of which facilitates accurate and uniform application. Allow the etching gel to react for 60 seconds.

Reduce etching times for enamel of juvenile teeth to approx. 30 seconds, extend to approx. 120 seconds for low-prism or fluoridated enamel. Then rinse thoroughly with sufficient water. Dry the surface to be sealed well. The etched tooth surfaces should be milky-lime white. After drying, contact with liquids should be excluded.

2. Apply only 1 drop of Fisseal to a mixing pad and apply a thin layer to the etched surfaces with a brush. Then cure with a halogen lamp for 20–30 seconds. Light devices should emit at 450 nm at 500 mW/cm² and should be checked regularly. Place the light as close as possible to the sealer. Close carefully after use.

Warnings**Residual risks**

The processing instructions and precautionary measures described in the instructions for use of the product must be observed in any case. If, despite following these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in every case, describing the accompanying circumstances as precisely as possible.

Side effects

Undesirable side effects of this medical device, when the device is properly processed and used, are extremely rare. Should any adverse effects occur – even in doubtful cases – please inform us, describing the accompanying circumstances and symptoms as precisely as possible. We follow up on every indication.

No risks in the event of an overdose are known of.

Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage

The Product should be stored between 0° C - 22°C.

Minimum shelf life

Do not use after the expiry date.

Variants

[REF] WP4170 Fisseal, white

2 syringes x 2,0 ml + 1 syringe x 2,0 ml Extra-Gel + cannulas

Disposal

Unused material can be disposed of cured or uncured as methacrylate waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

H315 causes skin irritation
H319 causes serious eye irritation

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