Extra Gel®

- Seite 2 - de: Gebrauchsanweisung
Page 3 - en: Instructions for use
Page 4 - fr: Manuel d'instruction
Página 5 - es: Instrucciones de uso
Pagina 6 - it: Istruzioni per l'uso
Strana 7 - bs: Uputstvo za upotrebu
Pagină 8 - ro: Instrucțiuni de utilizare
Sayfa 9 - tr: Kullanma kılavuzu
Стр. 10 - ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended user

Dentist

Composition

Qualitative data:

Phosphoric acid, glycerine, water, silicon dioxide, dye

Quantitative data in % G/G:

Phosphoric acid 37, glycerine > 20, water < 10, silicon dioxide > 5

Intended purpose of the product

Extra Gel is used to etch the tooth structure in the enamel etching technique.

Indications

The product is used for etching the tooth structure in the enamel etching technique.

Contraindications

May dissolve underfillings if left on for a long time. Do not use on acid-soluble underfillings.

Performance characteristics of the product

The etching of the tooth structure and, if necessary, the underfilling.

Clinical benefit

Extra Gel is a thixotropic etching gel containing 37% phosphoric acid. Easy-to-use packaging in a 2 ml syringe.

Specific precautions

Wear protective goggles.

Application

Preparation of the etching gel syringe

Remove the cap from the syringe and attach one of the supplied cannulas. If necessary, the cannula can be bent a little to facilitate application. CAUTION! Before the initial application, a little etching gel should be squeezed out of the syringe to prevent the etching gel from "shooting out" due to air pockets in the syringe.

Application

Apply the etching gel only to the enamel, avoid contact with the dentine and the edges of the tooth. Leave the etching gel to act for at least 15 seconds. For young teeth treated with fluoridation varnish, it is recommended to extend the exposure phase to 90-120 seconds. The etching gel is then rinsed off with running water for approx. 20 seconds. Then dry the applied area with the air blower using oil- and water-free air. The etched enamel should then show a milky-lime-white colour.

Processing instructions

The working instructions and precautionary measures described in the package leaflet accompanying the product must be observed in any case. If, despite adherence to these instructions, an adverse reaction occurs in the patient, or if other adverse events occur, we request that you inform us in any case, describing the accompanying circumstances as precisely as possible.

Processing the adhesive

Apply the adhesive of your choice with a brush to the saliva-free and dry etched enamel surface. Follow the instructions for use for the adhesive.

Placement, curing and finishing

Follow the manufacturer's instructions for use for the material in question regarding placement, curing and finishing.

Precautions

It is of the utmost importance that the etched enamel has not been otherwise contaminated. If there is contamination with other liquids, re-etch, then rinse again with running water and dry as described above. Wear suitable protective clothing, as well as gloves and safety goggles. It is absolutely essential to avoid contact between caustic gel and mucous membranes, the eyes and skin. Should such contact occur accidentally, clean the area immediately with sufficient running water.

Side effects

No side effects have been reported to date. In individual cases, hypersensitivity to components of the material may occur. Avoid skin contact with the paste. In case of accidental contact, wash immediately with water and soap. In case of eye contact, rinse immediately with plenty of water and consult a doctor. Should you become aware of any undesirable side effects – even in cases of doubt – please inform us in any case, describing the accompanying circumstances and symptoms as precisely as possible.

Risks of overdose are not known.

Interactions

Interaction of the product and its metabolites with other medical devices, medicinal products or other substances is not known.

Storage

Storage time 3 years at 22°C

Minimum shelf life

Do not use after the expiry date.

Variants

 WP4050 1 x 2.0 ml syringe + needle

 WP4051 2 x 2.0 ml syringe + needle

Disposal

Dispose of unused material.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

We know how

 **dent a pharm**
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

 **Willmann & Pein GmbH**
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

Made in Germany

H314 Causes severe skin burns and eye damage

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