Competence universal[®]

- Seite 2 - de: Gebrauchsanweisung
Page 3 - en: Instructions for use
Page 4 - fr: Manuel d'instruction
Página 5 - es: Instrucciones de uso
Pagina 6 - it: Istruzioni per l'uso
Strana 7 - bs: Uputstvo za upotrebu
Pagină 8 - ro: Instrucțiuni de utilizare
Sayfa 9 - tr: Kullanma kılavuzu
Strona 10 - pl: Instrukcja użytkowania
Стр. 11 - ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentist

Composition

Qualitative data:

oligomeric methacrylates, silicon dioxide, glass ceramics

Quantitative data in % G/G:

Oligomeric methacrylates 23, fillers midi 4, mini 65 and nano 8

Intended purpose of the product

Competence Universal is a light-curing, radiopaque universal filling composite.

Indications

Restoration of all cavity classes.

Contraindications

Use of Competence Universal is contraindicated if drainage or the recommended application method is not possible. Do not use directly on opened pulp or if there is a known allergy to any ingredient.

Performance characteristics of the product

Competence Universal is a universal filling composite that cures with light.

Specific precautions

None

Application

1. Clean the tooth before preparation, remove all cleaning residues with water spray. Determine the Vita® tooth colour. Using a dental dam is recommended.
2. Prepare the cavity according to the general rules of adhesive technique. Bevel the edges of the enamel and protect areas close to the pulp.
3. If required, etch according to your preferred technique, e.g. Total-Etch. Observe the instructions for the materials used.
4. Apply adhesion promoter (e.g. C-Bond)
5. If necessary, place an underfill of thin flowing composite (Competence Flow) or Compomer. Using a die is recommended.
6. Fill the cavity in the desired colour with layers no thicker than 2 mm. Leave each layer to light cure for 20 seconds.
7. Remove excess, contour and finish e.g. with diamond finishers and flexible discs of decreasing grain size. A high-gloss finish can be achieved with polishing pastes (e.g. Vision Paste) and polishing pads/discs.

Processing instructions

The processing instructions and precautionary measures described in the instructions for use accompanying the product must always be observed. If, despite adherence to these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in any case, describing the accompanying circumstances as precisely as possible.

Warnings**Residual risks**

Please observe carefully

- Light units should be emitting at 450 nm and should be checked regularly. Place the light as close as possible to the filling material.
- Do not remove the oxygen-inhibited layer, as this is needed for bonding with the next layer.
- Follow the instructions for use of other materials used with Competence Universal.

NOTE

The application aids included with the product (cannulas, disposable brushes, etc.) are only for single use on a single patient!

Side effects

No side effects have been reported to date. In individual cases, hypersensitivity to components of the material may occur.

Avoid skin contact with the paste. In case of accidental contact, wash immediately with water and soap. In case of eye contact, rinse immediately with plenty of water and consult a doctor. If you become aware of any adverse effects – even in doubtful cases – please inform us and describe the accompanying circumstances and symptoms as precisely as possible.

Risks in case of overdose are not known.

Interactions

Avoid materials containing eugenol, moisture and oily air, as they can prevent polymerisation at the point of contact. Discolouration may occur when using cationic mouthwashes, plaque disclosers or chlorhexidine.

Storage

Store in a dry place. The Product should be stored between 0° C - 22°C After use, empty the syringes immediately so that the material is not hardened by the ambient light.

Minimum shelf life

Do not use after the expiry date.

Variants

- [REF] WP4250: Competence universal, Set of four 4.5 g syringes in various Vita® colours (A2, A3, A3.5, B2), 10 ml C-Bond, 2 ml Extra Gel, 2 ml Glass Liner + accessories
- [REF] WP4251: Competence universal, Set of four 4.5 g syringes in various Vita® colours (A2, A3, A3.5, C2), 10 ml C-Bond, 2 ml Extra Gel, 2 ml Glass Liner + accessories
- [REF] WP4346: Competence universal, Set of four 4.5 g syringes in various Vita® colours (A2, A3, A3.5, A3 opaque), 10 ml C-Bond, 2 ml Extra Gel, 2 ml Glass Liner + accessories
- [REF] WP4388: Competence universal, Set of four 4.5 g syringes in various Vita® colours (A2, A3, A3.5, A2 opaque), 5 ml C-Bond, 2 ml Extra Gel, 2 ml Glass Liner + accessories
- [REF] WP4252: Competence universal, Refill, A1, 4.5 g
- [REF] WP4253: Competence universal, Refill, A2, 4.5 g
- [REF] WP4254: Competence universal, Refill, A3, 4.5 g
- [REF] WP4255: Competence universal, Refill, A3.5, 4.5 g
- [REF] WP4088: Competence universal, Refill, A4, 4.5 g
- [REF] WP4294: Competence universal, Refill, B1, 4.5 g
- [REF] WP4256: Competence universal, Refill, B2, 4.5 g
- [REF] WP4264: Competence universal, Refill, B3, 4.5 g
- [REF] WP4257: Competence universal, Refill, C2, 4.5 g
- [REF] WP4266: Competence universal, Refill, C3, 4.5 g
- [REF] WP4265: Competence universal, Refill, D2, 4.5 g
- [REF] WP4258: Competence universal, Refill, D3, 4.5 g
- [REF] WP4260: Competence universal, Refill, A2 opaque, 4.5 g
- [REF] WP4259: Competence universal, Refill, A3 opaque, 4.5 g
- [REF] WP4261: Competence universal, Refill, Incisal, 4.5 g
- [REF] WP4267: Competence universal, Refill, B2 opaque, 4.5 g
- [REF] WP9055: Competence universal, Refill, BXL, 4.5g
- [REF] WP4349: Competence universal, Set, 1 x A3, 4.5 g, 1 x 10 ml C-Bond
- [REF] WP4397: Competence universal, Refill, Bleach, 4.5 g
- [REF] WP4391: Competence universal, Refill, 0M1, 4.5 g
- [REF] WP4392: Competence universal, Refill, 0M2, 4.5 g
- [REF] WP4393: Competence universal, Refill, 0M3, 4.5 g

Disposal

Unused material can be disposed of with household waste after curing.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

H317 May cause allergic skin reactions
H318 Causes serious eye damage
H411 toxic to aquatic organisms, with long-lasting effect

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