Vision Prophy Powder®

Seite 2 - de: Gebrauchsanweisung
Page 3 - en: Instructions for use
Page 4 - fr: Manuel d'instruction
Página 5 - es: Instrucciones de uso
Pagina 6 - it: Istruzioni per l'uso
Strana 7 - bs: Uputstvo za upotrebu
Pagină 8 - ro: Instrucțiuni de utilizare
Sayfa 9 - tr: Kullanma kılavuzu
Стр. 10 - ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended user

Dentist, dental prophylaxis assistant

Composition

Qualitative data:

Sodium hydrogen carbonate, amorphous silica, sweetener and flavouring

Quantitative data:

- Sodium hydrogen carbonate 97.3%
- Amorphous silica 1 %
- Sweetener 1.6%
- Flavouring 0.1%

Intended purpose of the product

Cleaning the tooth surface

Indications

For cleaning only the tooth surface (supragingival)

Contraindications

- Patients and practitioners who suffer from chronic bronchitis or asthma must not be treated with or use an air-polishing device under any circumstances. The spray of air and powder could cause respiratory problems.
- Do not use on patients who have upper respiratory tract infections.
- Do not use on patients on a low-salt diet as the powder contains sodium bicarbonate.
- Do not use in case of allergies to the ingredients.
- Do not aim the powder spray at fillings, crowns or bridges as this could damage them.
- Do not use powder subgingivally

Performance characteristics of the product

Cleans the tooth surface

Specific precautions

- Patients and practitioners must wear eye protection
- The practitioner must wear a mask and goggles.
- Use long dental cotton rolls in the patient's mouth for maximum protection.

Application

Apply Vaseline to the patient's lips. Fill the powder into the air-polishing device and follow the manufacturer's instructions. Use the large suction cannula on your dental device.

The product may only be used by specialists (dentists).

Note:

The maximum powder filling level indicated on the powder container must not be exceeded.

If the powder bottle is left open for too long, the powder may clump and the device may become clogged. Large nozzles should be used with the device to be used in order to avoid blockages.

Warnings Residual risks

The working instructions and precautionary measures described in the package leaflet accompanying the product must be observed in any case. If an adverse reaction occurs in the patient or other adverse events occur (despite following these instructions), we request that you inform us in every case, describing the accompanying circumstances as precisely as possible.

Side effects

No side effects have been reported to date. In case of eye contact, rinse immediately with plenty of water. If you become aware of any adverse effects (even in doubtful cases), please inform us and describe the accompanying circumstances and symptoms as precisely as possible. Risks in case of overdose are not known.

Interactions

No interaction of the product and its metabolites with other medical devices, medicinal products or other substances is known of.

Storage

The powder should not be stored near acids or heat sources. Store between 0 and 22°C. Protect from moisture.

Minimum shelf life

Do not use Vision Prophy Powder after the expiry date.

Variants

 WP4913 Black raspberry 1 x 500 g

 WP4914 Mint 1 x 500 g

 WP4933 Lemon 1 x 500 g

Disposal

Dispose of unused material according to regulations.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

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Willmann & Pein GmbH