Vision Prophy Paste®

Seite 2	- de:	Gebrauchsanweisung
Page 3	- en:	Instructions for use
Page 4	- fr:	Manuel d'instruction
Página 5	- es:	Instrucciones de uso
Pagina 6	- it:	Istruzioni per l'uso
Strana 7	- bs:	Uputstvo za upotrebu
Pagină 8	- ro:	Instrucțiuni de utilizare
Sayfa 9	- tr:	Kullanma kılavuzu
Strona 10	- pl:	Instrukcja użytkowania
Стр. 11	- ru:	Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended user

Dentist, dental technician

Composition

Qualitative data:

Pumice stone powder, glycerine, calcium sulphate, aluminium silicate, polysaccharides, flavouring, colouring agent

Quantitative data:

60 % glycerine, 33 % pumice powder

Intended purpose of the product

Vision Prophy Paste is a universal polishing paste based on pumice powder.

Indications

- Cleaning before tooth whitening
- Professional polishing of teeth
- Polishing of plastics and precious metals
- Polishing of crowns and filling materials

Contraindications

Do not use in patients with known allergy to the ingredients.

Performance characteristics of the product

Smooths surfaces (enamel, plastics, metals).

Specific precautions

None

Application

Open the can end and remove the required amount of paste. Immediately close the can again. Put a little paste on a brush or polishing cup. Do not mix with water. Apply paste to tooth surfaces or materials to be polished. Work at as low a speed as possible. After polishing, have the patient rinse thoroughly with water. Clean polished materials with water.

Warnings**Residual risks**

The patient and the dentist are advised to wear protective goggles. Use product only for the specified indication areas. In the case of pre-treatment for tooth whitening (bleaching), no paste should be left on the tooth surfaces, as this can impair the bleaching effect.

Side effects

No side effects have been reported to date. In case of eye contact, rinse immediately with plenty of water. If you become aware of any adverse effects (even in doubtful cases), please inform us and describe the accompanying circumstances and symptoms as precisely as possible.

Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage

The Product should be stored between 0° C - 22°C . Should sedimentation occur or the paste dry out, this will not affect the effectiveness of the product.

Minimum shelf life

Do not use after the expiry date.

Variants

[REF] WP6120 Vision Prophy Paste Cinnamon, 1 x 160.0 g

[REF] WP6139 Vision Prophy Paste Strawberry, 1 x 160.0 g

[REF] WP6140 Vision Prophy Paste Lemon, 1 x 160.0 g

Disposal

Unused material can be disposed of with household waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

Made in Germany

H317 may cause allergic skin reactions

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