Glass Liner®

- Seite 2 - de: Gebrauchsanweisung
Page 3 - en: Instructions for use
Page 4 - fr: Manuel d'instruction
Página 5 - es: Instrucciones de uso
Pagina 6 - it: Istruzioni per l'uso
Strana 7 - bs: Uputstvo za upotrebu
Pagină 8 - ro: Instrucțiuni de utilizare
Sayfa 9 - tr: Kullanma kılavuzu
Strona 10 - pl: Instrukcja użytkowania
Стр. 11 - ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentist

Composition

Qualitative composition: Glass ceramic, glass ionomer powder, silica, camphorquinone, hexanediol dimethacrylate, Bis-GMA, BHT, DMTBA
Quantitative composition: Fillers 65 %; activators, accelerators and stabilisers 1 %, dimethacrylates 34 %

Intended purpose of the product

Glass Liner is a light-curing, radiopaque, one-component underfilling material.

Indications

Underfilling material for composite restorations

Contraindications

The material contains methacrylates.
Do not use in patients with known allergy to the ingredients.

Direct or indirect protection of the pulp must not be carried out with products containing eugenol, as otherwise the complete curing of Glass Liner with LED light cannot be carried out. There is no restriction of the filling material over the cured glass liner.

Performance characteristics of the product

Glass Liner polymerises with light to form a solid and compact polymerisate.

Specific precautions

None

Application

- Clean the tooth before preparation, remove all cleaning residues.
If the dentin above the pulp is smaller than 0.5 mm (shining through pulp), a pulp protection (calcium hydroxide preparation) must always be provided to neutralise possible clinically relevant diffusion of acrylic components.
- Prepare the cavity according to the general rules of adhesive technique. Protect pulp in deep cavities with suitable materials.
- Etch the cavity according to the preferred technique (e.g. total-etch). Observe the manufacturer's information for the materials used.
- Apply adhesion promoter.
- Just before application, apply the necessary amount of material on a mixing block. Protect from daylight and operating light to prevent premature hardening of the material. Apply a thin layer of the underfilling material with a small, round condenser. The layer thickness should not exceed 1 mm. If a thicker layer is necessary, work in layers and cure the layers one after the other. The material must only be applied to dentin and not to enamel. Remove excess material and then cure with a LED lamp (at 450 nm with a light intensity of at least 400 mW/cm²).
- To do this, bring the light source as close as possible to the underfilling material. At a distance of approx. 2 mm from the surface, the underfilling should be cured for at least 30 seconds. Never cure for less than 20 seconds. Any stubborn residue on the enamel should be removed with a rotary instrument.
- Place the filling material (composite) according to the manufacturer's information.

Addendum

After curing, the underfilling is so hard that the remnants of the material can no longer be removed with a probe. This would loosen the whole underfilling.

Store at room temperature and not above 22°C. Cooling the material (4-8°C) increases the shelf life. Before use, however, it should be brought back to room temperature.

Warnings**Residual risks**

The product contains methacrylates.
Persons who have an allergy to methacrylates should not come into contact with the product.

Side effects

Adverse side effects are not known. Risks in case of overdose are not known.

Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage

The Product should be stored between 0° C - 22°C.

Minimum shelf life

Do not use Glass Liner after the expiry date.

Variants

-  WP5601 1 x syringe with 2.0 ml + cannulas
 WP5602 2 x syringes with 2.0 ml + cannulas
 WP5680 Glass Liner (TR) 1 x Spritze mit 2,0 ml + Kanülen
 WP5681 Glass Liner (TR) 2 x Spritzen mit 2,0 ml + Kanülen

Disposal

Leftovers and contaminated packaging should be disposed of as hazardous waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

Made in Germany

H315 Causes skin irritation
H335 May cause respiratory irritation
H319 Causes serious eye irritation

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