Securafil LC®

- Seite 2 - de: Gebrauchsanweisung
Page 3 - en: Instructions for use
Page 4 - fr: Manuel d'instruction
Página 5 - es: Instrucciones de uso
Pagina 6 - it: Istruzioni per l'uso
Strana 7 - bs: Uputstvo za upotrebu
Pagină 8 - ro: Instrucțiuni de utilizare
Sayfa 9 - tr: Kullanma kılavuzu
Strona 10 - pl: Instrukcja użytkowania
Стр. 11 - ru: Инструкция по применению

We know how

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Patientenzielgruppe

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Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentists

Composition

Qualitative data:

Powder: Calcium-aluminium-fluorine-silicate, polyacrylic acid, iron oxide

Liquid: Oligomer methacrylate, light activators, inhibitors, accelerators

Quantitative data:

Powder: > 90% calcium-aluminium-fluoride-silicate, polyacrylic acid > 5%

Liquid: Methacrylate > 98%, additives > 1%

Intended purpose of the product

Securafil LC is a light-curing 2-component glass ionomer filling material for cavity classes III and V.

Indications

For filling cavities of class III and V.

Contraindications

Do not use on patients with known allergies to the ingredients.

Performance characteristics of the product

Replacement of the tooth structure with a self-curing and light-curing glass ionomer cement.

Clinical benefits

Replacement of missing tooth structure to restore function.

Specific precautions

None

Application

Mixing

- Shake the powder before dispensing it.
- One level measuring spoon is used with one full drop of liquid (holding the bottle vertically, without air bubbles, squeezing out drops with slight pressure).
- Mixing is carried out first with half of the powder, then with half of the remaining powder, and so on until the consistency required is reached.

Application

- Clean the cavity thoroughly and dry well.
- Mix the quantity required just before application.
- Protect from daylight and surgical lighting, as this light can cause premature curing.
- Place the filling material in the cavity in layers with a suitable instrument.
- The layers should be no thicker than 2mm and should be polymerised layer by layer.
- Cure the material with a halogen lamp. Bring the light source as close as possible to the material. At a distance of 2 cm from the surface, the filling takes at least 40 seconds to cure.
- Any excess material on the enamel can be removed with a rotating instrument.

Warnings**Residual risks**

Some patients may react with hypersensitivity to components of the product. Product must not be used during pregnancy.

Side effects

Adverse side effects from this medical product are extremely rare if used correctly. In rare cases, components of Securafil LC may cause sensitisation. If you become aware of any effects – even in doubtful cases – we ask that you always inform us and describe the accompanying circumstances and symptoms as precisely as possible.

There are no known risks in the event of an overdose.

Interactions

Interaction of the product with other medical products, pharmaceuticals or other substances is not known.

Storage

Store in a dry place, protected from sunlight, store at room temperature, not above 22°C.

Minimum shelf life

Do not use after the expiry date.

Variants

REFWP4144 - 1 x 15,0 g powder, A2, 1 x 8,0 g, liquid + measuring spoon

REFWP4124 - 1 x 7,0 g powder, 1 x 5,0 g, liquid + measuring spoon

Disposal

Hardened residues can be disposed of with household waste.

Any serious incidents relating to the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is based.

We know how

dent a pharm
Produktionsgesellschaft mbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9225-0
Fax: +(49) 4123/9225-48
www.dent-a-pharm.de
info@dent-a-pharm.de

Willmann & Pein GmbH
Schusterring 35
25355 Barmstedt/Germany
Fon: +(49) 4123/9228-0
Fax: +(49) 4123/9228-49
www.wp-dental.de
info@wp-dental.de

Made in Germany

Attention
Warning

H317 may cause allergic skin reactions

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